novafon®

2500054-99_V01 – Stand: 09/2025

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1. Key for overview	29
2. Preface	29
3. Product description	29
3.1. Scope of delivery	29
3.2. Device type, technical specifications	29
3.3. Accessories	30
3.3.1. Disc attachment	31
3.3.2. Ball attachment	31
3.3.3. Charging stand	31
3.3.4. Optional accessories	31
3.3.5. Combination with other products	32
4. Operation	32
4.1. The Display	32
4.4. Charging	33
4.5. Attachments	34
5. Application & Use, User Groups	34
5.1. Intended use	35
5.2. General information for use	35
5.3 Self-application at home	36
5.4. Professional application in neuro-rehabilitation	37
5.4.1. Spasticity	37
5.4.2. Hemineglect	37
5.5. Professional use in voice and swallow therapy	38
6. Contraindications and side effects	38
7. Technical specifications	39
7.1. Symbols used	39
7.2. Technical specifications	40
8. Important safety instructions	41
8.1. Warnings	42
8.2. Other precautions	45
9. Maintenance and repair	47
9.1. Cleaning and disinfection	47
9.2. Troubleshooting	47
9.3. Safety check, maintenance and repair	48
9.5. Disposal	49
10. Warranty	49
28	

11. Customer service	50
12. Simplified EU Declaration of Conformity 50	

1. Key for overview

On page 1 you will find an overview of your device with the following important elements:

- 1 Display Intensity
2. Multi-button
 - top: intensity setting in three levels
 - bottom: frequency setting Standby / ON / 100 Hz / 50 Hz / OFF
3. Frequency indicator
 - 1 illuminated dot (): 100 Hz
 - 2 illuminated dots (): 50 Hz
4. Battery / charging status indicator
5. USB charging port
6. Correct positioning of the attachment on the device

2. Preface

Congratulations on your purchase of a novafon vibration sound wave device. Our therapy devices have delivered excellent performance and reliability for decades. Please read the following instructions carefully to ensure that you can get the most out of your novafon vibration sound wave device and enjoy using it for a long time. The explanation of the footnotes can be found on the last page of this instructions.

3. Product description

3.1. Scope of delivery

Delivery includes a battery powered novafon vibration sound wave device as well as a disc attachment, a ball attachment, a power plug, a charging cable and instructions for use.

Intraoral attachments are included in some models. In these cases, they are part of the standard delivery of the devices but are also available as spare parts.

3.2. Device type, technical specifications

novafon vibration sound wave devices are designed for local vibration therapy. Different attachments are used to transmit the mechanical vibrations into the tissue. Thanks to its safe and simple application, local vibration therapy can be used in many ways.

Thanks to the two treatment frequencies and three intensity settings treatment can be individually adapted to the body part and underlying symptoms in question.

The following table describes the characteristics of your new novafon device

Device type	novafon one
Multifunction buttons	ON/OFF (⏻) and (⏮)
frequency settings (⏻)	50 Hz, 100 Hz, Standby, Off
Intensity / strength setting (⏮)	low (⏮), medium (⏮), high (⏮)
Intensity	●●○○○
Depth of penetration	6 cm +
Colour	white
Lithium ion battery	Capacity 2400 mAh, voltage 3,6 V, 8,8 Wh, Type INR18650 with protection board
Charging time	approx. 3h
Battery life	up to 3.5h depending on frequency and intensity

For more technical specifications please see the technical data sheet in the download section of our website.

3.3. Accessories

novafon vibration sound wave devices are equipped with two interchangeable attachments as standard. NOVAFON GmbH also offers other attachments that are not included in this set. Please contact your dealer or visit www.novafon.com for more information. novafon devices may only be used with original novafon accessories.

3.3.1. Disc attachment

The disc attachment is used in the treatment of larger body parts that require broader strokes. It is supplied as standard and also available as a spare part.

3.3.2. Ball attachment

The ball attachment can be used for smaller areas and targeted treatment. It is supplied as standard and also available as a spare part.

3.3.3. Charging stand

The charging stand can be purchased together with the device or separately. The charging stand offers convenient storage for your device during and after charging. These instructions for use include all the information you need on how to use and clean your stand, as well as any necessary precautionary measures.

3.3.4. Optional accessories

On our website you will find additional attachments that simplify treatments in different areas of the body.

Note: Depending on your country, the device may also be equipped with a second power plug with country-specific connections. These are also available as spare parts.

The magnetic head attachment accessory is designed for targeted application, for example, to treat trigger points. Its shape focuses the sound waves in a special way, allowing for particularly deep penetration. It does not have its own instructions for use. The information on changing attachments and cleaning in these instructions for use also applies to the magnetic attachment. The attachment is a Class I medical device in its own right.

3.3.5. Combination with other products

novafon devices are not intended to be used with medical products other than novafon original accessories.

THE USE OF ACCESSORIES FROM OTHER MANUFACTURERS CAN IMPAIR THE FUNCTION AND PERFORMANCE OF NOVAFON DEVICES.

4. Operation

4.1. The Display

The current operating status is displayed by the 3 arcs above the button and the 2 points below the button. If none of the indicators are lit, the device is off/stationary or the battery is completely empty. The two dots show the currently selected frequency (1 point = 100 Hz, 2 dots = 50Hz). The three arcs the currently selected intensity (low, medium, high). The single dot on the bottom end of the handle flashes when the battery level is low or when the device is charging.

4.2. Initial start-up and frequency

Fully charge the device prior to switching it on for the first time (chapter 4.4).

The device starts at 100 Hz (•) by default. Pressing the button again decreases the frequency to 50 Hz (••). The current frequency selection is indicated by the LED display (frequencies). Press the button a third time to switch off the device.

4.3. Setting the intensity / strength

You can regulate the intensity of your device by repeatedly pressing the intensity button. Here you can choose from three intensity settings: low (≡), medium (≡≡), and high (≡≡≡). You can recognize the intensity selection by the LED display (intensity).

The device is set to start on the lowest intensity setting by default (≡). This also corresponds to novafon's recommendations for treatment i.e. start at low intensity and increase as required, according to each individual's sensitivity.

NOTE: The best vibration is achieved when the device is operated without additional contact pressure. If your device automatically switches to a lower setting, or you are unable to increase the intensity, this is probably due to a low battery level. In this case, you should charge your device.

4.4. Charging

To charge the device, you should only use the supplied power plug and cable, or the novafon charging stand if included (depending on the model, the charging stand may not be included in delivery). To charge your device, first connect the cable to the device or charging stand and then to the power plug suitable for your power connection. Only then should you plug the power plug into the socket. To charge the device without a charging stand, use the charging cable included in the device packaging. Charge the device until the battery is fully charged. Do not use the novafon while it is charging. The device has a low battery indicator.

When using the charging stand to charge your device, make sure you are careful when placing the device onto the connector and removing it. To remove, hold the charging stand and gently release the device. The device is charging when the battery indicator appears (see view 1).

Avoid twisting the device in the charging station in order to prevent damage.

To avoid malfunctions, the device must be completely dry and free from residues of cleaning agents, disinfectants or oils / creams.

Note: When the device is fully charged, disconnect the power plug and device from the electrical supply to avoid damage. If the device is connected to the charging accessory, all pole breaking and complete disconnection from the mains can only take place if the mains plug is removed from the socket.

4.5. Attachments

The attachments are fixed to the device via a plug connection. They can easily be detached by turning them a quarter turn and pulling them gently off the device. The attachments click simply onto the device. The attachments are in the correct position when the brackets on the device and the attachment lie seamlessly on top of each other. The magnets help you to find the final position. Before starting treatment, make sure that the respective attachment is engaged in the intended final position. Please refer to the drawing on page 1 for the correct position of the attachment on the device.

5. Application & Use, User Groups

novafon sound wave devices are used by doctors and therapists to support their treatment in occupational therapy, physio therapy and speech therapy. Thanks to their ease of use, the devices can also be used in

the home environment (home use by lay users). Training in how to use the devices is therefore not required. The information in these instructions for use must be observed. Please note that novafon sound wave devices only alleviate symptoms. They do not treat the underlying cause and are not a substitute for visiting a doctor. Despite their safe, simple, and non-invasive application, we suggest that you ideally discuss the treatment with your treating doctor or therapist. If symptoms worsen, please seek medical advice. Stop the treatment immediately if you feel dizzy, unwell, or feel skin irritation or increased pain. Please always note the contraindications listed in chapter 6. Please note that some applications may require the use of special novafon accessories (sold separately). Devices and accessories can be used from head to toe. **DO NOT USE ON THE GENITALIA.**

novafon devices can be used on patients from the age of 6.5 Use on younger patients may also be possible if monitored by specialist staff. However, **DO NOT** use novafon devices on patients under the age of 2.

5.1. Intended use

novafon devices and accessories are designed to relieve pains and aid in relaxation of muscles¹ to alleviate symptoms of:

- Temporary and transient pain of the musculoskeletal system
- Chronical pain of the musculoskeletal system¹
- Neuro Rehabilitation (sensorimotor disorders)^{2,3}
- Voice and swallowing disorders⁴

Professional users: Occupational therapists, physio therapists, speech therapists and the like. Home use by lay users, mostly in self-application, but also relatives and caretakers to mobility impaired patients

5.2. General information for use

The different frequencies have different effects, so 100 Hz¹ is used in particular to reduce pain, but also to activate muscles. 50 Hz², on the other hand, serve to detonate the muscles and thus help in general relaxation, but also in relaxation of spastic muscles.

Treatment should not exceed a maximum of 20 minutes per session, nor a maximum of 3 sessions per day. Please note that after 20 minutes of use, the novafon should be switched off for 15 minutes in order to avoid overheating.

When treating severely painful or sensitive parts of the body (e.g. face, neck, head), you should reduce / adjust the duration of treatment according to each individual's sensitivity.

Note: Application of liniments: Apply only a thin layer of creams, ointments or cosmetic / medicinal liniments to the skin, or place a fine cloth between the device and the skin. Do not allow liquids or residues to penetrate the novafon as this may cause damage. After using agents of this type, make sure you clean the device thoroughly.

Note: Discontinue treatment immediately if side effects occur (chapter 6).

5.3 Self-application at home

Taking into account the contraindications (chapter 6), move the novafon device in a straight line, selectively or in slowly circling movements on or around the treatment area. The novafon device can also be applied at trigger points. In case of uncertainty, please consult your treating physician or therapist regarding your individual application. The novafon device can be used directly on the skin or through fabric / clothing. During the application, make sure that

the vibration sound wave device glides gently and without pressure over the body area. Use the entire surface of the attachment. This is the only way the penetration depth effect of the novafon device can be fully deployed and the vibrations are transmitted optimally. Use only the novafon device's own weight for a comfortable application.

More detailed information on possible treatment sequences can be found in our application book (enclosed with the delivery or online in our download area at www.novafon.com)

5.4. Professional application in neuro-rehabilitation

Note: The use of local vibration therapy for neurological diseases should only be carried out by specialists (therapists, physicians, nurses) or after detailed instruction by such a person. The general application descriptions from the previous chapters also apply to professional use.

5.4.1. Spasticity

Spasticity is a common consequence of a stroke and other disorders of the central and peripheral nervous system. Local vibration therapy, combined with physiotherapy, provides a simple and drugfree treatment method. Numerous studies have shown a reduction in spasticity as well as improved motor control and function.² The effectiveness mechanisms are the tonic vibrational reflex and the stimulation of the sensomotoric system. Taking into account the contraindications (chapter 6), the spastic muscle itself is treated at 50 Hz and the antagonist of the spastic muscle at 100 Hz. The starting point can be on the tendon or directly on the muscle itself. The treatment may be local, i.e. focusing on one point, or in sweeping movements. The application with the novafon device shall not exceed 20 minutes per unit. The treatment can be performed up to 5 times a week. Please keep

in mind that using the appliance incorrectly can lead to increased muscle tone. Stop the treatment immediately if any side effects occur (see chapter 6)

5.4.2. Hemineglect

Several studies have found an improvement in the visual exploration performance of stroke patients with Hemineglect when the neck musculature contralateral to the lesion is stimulated by vibration.³ This effect is based on the fact that the vibration stimulates stretch receptors in muscles and tendons of the neck, leading to an apparent rotation of the head. For successful therapy, the corresponding stimulation point must be found on the contralesional, posterior neck muscles, taking into account the contraindications (chapter 6). This point is determined by fixing a bright spot on the wall (e.g. by means of a laser pointer) in a darkened room. The therapist guides the novafon device slowly along the neck muscles at 100 Hz and medium intensity. The patient sits on a chair for the treatment. The correct stimulation point is determined as soon as the patient reports that the fixed point is moving without moving his / her head. The total treatment time should not exceed 40 minutes, whereby the pure stimulation time should be interrupted by a 5 minute break after 10 minutes. A total of at least 15 treatment sessions are necessary to achieve a lasting effect. Stop the treatment immediately if any side effects occur (see chapter 6)..

5.5. Professional use in voice and swallow therapy

Note: The use of local vibration therapy for swallowing and voice disorders is only to be carried out by specialist staff (therapists, doctors, nursing staff) or after detailed instruction by such a person. The general application descriptions from the previous chapters also apply to professional use.

6. Contraindications and side effects

Treatments with the novafon vibration sound wave devices are not permitted for patients with:

- Open wounds / eczema (in the body region to be treated)
- Arteriosclerosis (in the body region being treated)
- Current episodes of epilepsy
- Pregnancy
- Implants (in the body region being treated)
- Heart pacemakers and defibrillators
- Thromboses (in the body region being treated)
- Brain pacemakers and deep brain stimulators
- Cardiac arrhythmia
- Tumours, cancerous and not cancerous
- Acute inflammations
(in the body region being treated)
- Acute episodes of inflammatory diseases

Side effects:

Please note that the following side effects may occur:

- Increased pain/unpleasant sensation (occasionally)
- Discomfort/dizziness/restlessness (occasionally)
- Skin irritation (rarely)
- Bruises (very rarely)
- Reflexes/involuntary muscle contractions (rarely)
- Muscle soreness (rarely)

Note: Pain treatment in particular can lead to an initial worsening of the symptoms before you notice any improvement.

CAUTION: Discontinue treatment immediately if you experience any side effects. Consult a physician if symptoms persist.

Note: Especially with pain management, symptoms may initially worsen before you notice improvement.

Note: Slight skin redness and warmth are not a cause for concern. This is a desired effect of local vibration therapy, caused by increased blood flow.
Do not use to treat pain or swelling of unknown origin. In this case, consult a physician for evaluation and diagnosis.

7. Technical specifications

7.1. Symbols used

- ~ Alternating current (AC)
- ⎓ Direct current (DC)
-  Device with protection class II according to DIN EN 61140
- Hz Hertz
- KB Short-term operation
- mA Milliampères
- mAh Milliampères per hour
- A Amp
- V Volt
-  Disposal according to the EU Directive on Waste Electrical and Electronic Equipment (WEEE)
-  Displays the manufacturer of the medical device
-  Medical-device
-  Unique-device-identification for medical products
-  Protection class of the application part: type BF
-  Temperature limit: Refers to the temperature limits to which the medical device can be safely exposed
-  Humidity limit
-  Atmospheric pressure limit
-  Refer to the instructions
-  Use inside only
-  General warning symbols
-  Intensity button (3 intensities)
-  Frequency setting, Stand-by / ON / 100 Hz / 50 Hz / OFF

7.2. Technical specifications

- Operating conditions: 5 °C (41 °F) to 30 °C (86 °F); 15 % to 93 % relative humidity, non-condensing; 700 hPa to 1060 hPa air pressure
 - Storage / transport: -5 °C (23 °F) to 40 °C (104 °F); 15 % to 93 % relative humidity, non-condensing; 500 hPa to 1060 hPa air pressure
 - Operating mode: Short-term operation 20/15 min
 - Operating noise: maximum 70 dBA
 - Operating temperature: maximum 50 °C (122 °F) for maximum service life.
 - Power plug Model 1180004
 - Input power: AC 100–240 V, 50–60Hz, 300 mA
 - Output power: DC 5 V, 2 A
 - LOT: Shows the date of manufacture of the power supply
 - A type F (EU plug) power plug is included in the standard scope of delivery. Additional mains adapters with different power plugs are optionally available or already included in the corresponding country sets.
 - USB-A charging cable to spring contact pin, cable length: approx. 1 m
 - One charge of a new battery guarantees at least 6, 20-minute treatment sessions at medium intensity. Note that frequent use at the highest intensity level can decrease battery life
- Battery: Lithium-ion battery type INR18650, permanently installed, lifespan: 800 charging cycles
- Battery specifications: Capacity 2400 mAh, Voltage 3.7 V, 8.8 Wh; permanently installed, lifespan: 800 charging cycles
 - Protection class IP22: protected against finger contact; protected against dripping water falling at an angle (up to 15°)

Due to the IP rating and safety requirements for medical devices, it is not possible for the battery to be replaced by the end user.

- Devices comply with IEC 60601-1-2:2014/AMD1:2020 and / or CISPR 11 Emission Class B and Group 1. Devices have been tested for home use in accordance with interference immunity testing levels.
- Product category: Class IIa medical device pursuant to the EU Regulation on Medical Devices (2017/745/EU), conformity with 2015/863/EU is also declared.
- Manufacturer: novafon - Elektromedizinische Geräte GmbH (novafon GmbH), Daimlerstraße 13, 71384 Weinstadt, Germany

8. Important safety instructions

Before starting to use the device, please read the user manual and these safety instructions carefully and strictly follow them.

8.1. Warnings

When using an electrical device basic precautions should always be followed including the following to reduce the risk of burns, fires, electric shock, possibly severe injury and possible death to the user or other people:

⚠ WARNING: Do not use the novafon device during charging.

⚠ WARNING: Always store the device in a dry and clean location.

⚠ WARNING: Before connecting the power plug to the mains power source, check that the mains voltage corresponds to the voltage specification on the power plug. Connect the novafon device only to a power outlet that delivers the correct voltage.

⚠ WARNING: To charge the device, use only the charging cable and power supply included in the delivery or the separately available charging stand. Use the correct connector plug, if applicable. Non-observance

can lead to increased electromagnetic emissions or reduced electromagnetic immunity and thus to malfunctions or unexpected operational behaviour.

⚠ WARNING: Always unplug the device and all charging accessories from the electrical outlet after charging and before cleaning. Make sure that the mains plug is easily accessible at all times so that you can easily disconnect the device from the mains in an emergency. Do not place heavy objects or furniture in front of the power adapter, even when using the charging stand.

⚠ WARNING: Ensure that devices are completely dry and with no cleaning agent, disinfectant or oil/cream residues before using the charging stand or charging cable.

⚠ WARNING: Do not bring the device and charging accessories (plug, charging cable, charging stand) into contact with hot surfaces (e.g. heater, hotplate, etc.). The housing of the device or adapter or the insulation of the cable could be damaged.

⚠ WARNING: Do not use the device when taking a shower or bath.

⚠ WARNING: Do not use the device where aerosol (spray) products are used, in other explosive atmospheres, or anywhere where oxygen is administered.

⚠ WARNING: Do not place or drop the device into water or other liquids.

⚠ WARNING: Do not reach for a device that has fallen into water, especially when the charging cable is connected to a power outlet. Unplug immediately.

⚠ WARNING: Do not place or store the device where it could fall or be pulled into a tub, pool or sink.

⚠ WARNING: Protect the device, charger and charging stand from drop impact and shocks.

⚠ WARNING: This device may not be modified without the manufacturer's permission.

⚠ WARNING: novafon devices may only be used with original novafon accessories. The use of accessories from other manufacturers can impair the function

and performance of novafon devices. This also applies to the power adapter and charging cables.

⚠ WARNING: Do not use the device and either discard or return to your dealer for examination if any of the following conditions occur:

- if the device fails to work correctly
- if the device is damaged, e.g. after having been dropped
- if the device was immersed in water or other liquids
- if the device exhibits other safety-related problems, such as emitting smoke, unusual or excessive heat etc.

⚠ WARNING: In order to prevent adverse events to the patient and operator due to electromagnetic disturbances, the novafon shall not be operated outside its intended EMC environment. Furthermore, the novafon shall not be operated if the enclosure, cables or measures for electromagnetic shielding are damaged.

⚠ WARNING: Do not use the power plug, charging cable or charging stand:

- in the event of visible damage or after dropping the power plug, charging cable or charging stand
- if the power plug, charging cable or charging stand has been exposed to extreme moisture
- if liquid has penetrated the unit

⚠ WARNING: Never charge the device for more than 24 hours at a time. Do not leave the device unattended. Disconnect the device from the power supply at night. This also applies when using the charging stand.

⚠ WARNING: Do not place the device on flammable surfaces when charging.

⚠ WARNING: Never leave the device unattended when it is turned on.

⚠ WARNING: Do not clamp or pinch any part of your body, especially your fingers, between the attachment and the device. During the treatment, hold the device only by the device handle.

⚠ WARNING: Never drop or insert any foreign object into any opening on this device. Do not insert any objects (including wires) into openings in the device.

⚠ WARNING: Wireless communication devices (such as Wi-Fi stations, mobile phones, cordless phones, and related base stations) can affect the device and should be kept at least 0.5 m (2 feet) away from the device during use. This also applies to cables and plug-in power supplies.

⚠ WARNING: Careful supervision is necessary when used on or by children, immobile or disabled individuals. novafon devices should only be used by users from the age of 18 years who have read the instructions and are competent to use the device safely.

⚠ WARNING: Do not use on sleeping or unconscious individuals.

⚠ WARNING: Keep the device out of reach of children when not in use.

⚠ WARNING: Small parts (e.g. cover caps) could come loose and be swallowed. Risk of suffocation!

⚠ WARNING: Keep children away from packaging materials. There is a risk of suffocation.

⚠ WARNING: Do not use on genitals, this device is not intended as an intimate pleasure toy.

⚠ WARNING: Do not use outdoors or in wet or damp areas.

⚠ WARNING: Never operate the power plug or charging stand in a high-humidity environment or in the presence of moisture. Please note that moisture is also generated by e.g. nebulisers and paraffin baths.

⚠ WARNING: Do not use or charge the device under blankets, pillows, towels, clothing, etc. or in places where ventilation is restricted, as it may overheat and could cause fire, electric shock or other personal injury.

⚠ WARNING: Avoid contact with eyes.

⚠ WARNING: Avoid contact with mucous membranes unless using specially designed attachments.

⚠ **WARNING:** Use the device only for its intended use as described in this manual.

⚠ **WARNING:** Do not use the device together with other medical devices and do not stack the devices on such products, as this may affect the performance and function of the device as well as the other product.

⚠ **WARNING:** Avoid overheating the device. Do not use the device for more than 20 minutes per application. Allow the device to cool down for at least 15 minutes before switching it on again.

⚠ **WARNING:** For technical reasons, the device can reach a temperature of up to 50 °C, especially on the head. Therefore, do not hold the device by your head for long periods of time. Change your grip position if the device feels uncomfortably warm. Switch off the device if any part of it feels hot and allow it to cool down thoroughly before using it again.

8.2. Other precautions

Note: Use the initial session to find out how your body responds to vibration therapy. If you experience any discomfort, dizziness or nausea, stop the session immediately.

Note: Do not use the device if any of the contraindications described in chapter 6 apply to you.

Note: Do not use the device for more than 20 minutes per application. Leave it to cool down for at least 15 minutes before restarting.

Note: Make sure to secure the attachments fully onto the novafon device and keep it on during treatment.

Note: If you perform multiple treatment sessions in quick succession with a maximum duration of 20 minutes and only use a single attachment, the temperature of the attachment can reach up to 50 °C. You can avoid this by removing the attachment between

sessions (e.g. for cleaning) or by using alternate attachments within the session. Stop the treatment if the contact surface feels uncomfortably warm.

Note: Make sure that no substances, such as liquids, creams or the like, can penetrate into the device or the power plug.

Note: To prevent damage, make sure that the device and charging stand are standing or resting on a stable surface during the charging process.

Note: Do not twist the device in the charging stand. This may cause damage.

Note: Keep the device away from dust, lint and other particles. The penetration of dust, lint or other particles can cause the device to fail. Clean your device regularly with suitable agents to prevent any damage caused by contamination. Please note chapter 9.1.

Note: To prevent premature wear, only store the device in a dry and clean place that is protected against UV light (incl. sunlight).

Note: Use only non-alcoholic cleaning supplies. Alcohol can attack the surfaces of the housing and the attachments.

9. Maintenance and repair

9.1. Cleaning and disinfection

Do not clean the device during charging. Disconnect the device from the power supply when cleaning. If used privately, clean the housing and attachments using a clean cloth moistened with fresh water or a mild soapy solution. For clinical and professional use, clean and disinfect device housings and all attachments with an appropriate surface disinfectant suitable for medical devices before and after each use. As a general rule, only use alcohol-free disinfectants

that are suitable for use on sensitive surfaces, such as NOVACLEANER by novafon, SinAlc sensitive by HIMED or RHEOSEPT-WD plus wipes by Rheosol. If necessary, dry off with a soft cloth.

If you are using the charging stand, this can be cleaned with a clean cloth moistened with fresh water or a mild soapy solution if necessary.

For professional use: To prevent contamination, only place cleaned devices in the charging stand

Ensure that devices are completely dry and with no cleaning agent, disinfectant or oil / cream residues before using the charging stand or charging cable.

9.2. Troubleshooting

If your device automatically switches to a lower setting, or you are unable to increase the intensity, this is probably due to a low battery level. In this case, you should charge your device.

You can also use the FAQs section of our website to find helpful answers to frequently asked questions. This is also available in downloadable format. If other malfunctions should occur in the device, please contact our customer service department. Never open the device yourself to rectify faults. Any attempt to repair the device yourself will void your warranty. For this product, the manufacturer declares sole responsibility for conformity with the requirements for class IIa medical devices pursuant to the EU Regulation on Medical Devices (2017/745/EU). Please contact us if you suspect that the device has caused a serious incident. As a user, you have the right to report incidents of this type to the relevant national authority (e.g. Ministry of Health).

9.3. Safety check, maintenance and repair

For the novafon device, no regular inspection or maintenance in the sense of a safety check (e.g. according to MPBetreibV) is intended. To ensure safe operation,

however, you should inspect the device and attachments for obvious damage such as cracks before use. Attachments must be replaced as soon as they show clear signs of wear or damage (cracks, deformation, changes to the surface). Please note that national laws may nevertheless require you to perform safety checks on the device. Therefore, for professional use, e.g. in practices or hospitals, it is recommended to carry out an electrical safety inspection, e.g. according to DGUV V3 (German Social Accident Insurance), at least every 2 years. This generally does not apply to private individuals. If you experience problems with your device, you may dispose of it or return it to your dealer. Repairs can only be carried out by novafon or qualified dealers. Information on repair options and spare parts is available on our website. Due to cleaning requirements and the necessary IP rating, the battery cannot be replaced by the end user. The battery may only be replaced by novafon or an authorized technician.

Note: Attachments and batteries are wear parts and are therefore subject to limited warranty.

9.5. Disposal

Devices must never be disposed of with common household waste. For environmentally sound disposal, the user is obligated to dispose of the old device at a designated collection point for the disposal of electrical and electronic equipment. The device contains a permanently installed battery and must be disposed of accordingly. Please observe the national legal regulations when disposing of the device. You can also send your old device to novafon; we will gladly dispose of it for you. This device is subject to EU Directive on Waste Electrical and Electronic Equipment (WEEE). To ensure the conservation of resources, please dispose of the packaging separately as waste paper.

10. Warranty

From the date of delivery, we grant a warranty of 2 years (24 months) for defects resulting from material or production defects. Warranty coverage is subject to the presentation of a proof of purchase. The battery, charging cable, power plug and the attachments is guaranteed for 6 months. Excluded from the warranty: Damage due to improper use, normal wear and tear and consumption (also valid for accessories). The right of rescission (reversal of the purchase), reduction (discount on the purchase price) or subsequent delivery is initially excluded. However, you have a claim to reduction or conversion at your discretion if we do not remedy a defect that is subject to our warranty within a reasonable period of time. Any damage caused by failure to use the product in accordance with its intended use or resulting from improper or unreasonable use or maintenance is not covered by the warranty. Such damage is the sole responsibility of the user. Failure to comply with this instruction manual will void the warranty. Some states may not allow such limitation, so it may not apply to you. This limited warranty gives you specific and you may have other rights which may vary from state to state. Get in touch with your dealer or supplier to claim warranty.

11. Customer service

The device may only be repaired by novafon Customer Service. Please do not send in your defective device without being asked, but rather clarify the damage and repair costs with us first. Repair could be more expensive than a new device. If the repair is no longer financially feasible within the warranty period, novafon offers a replacement unit (again with a 24-month warranty). This manual can also be viewed as a PDF file at www.novafon.com.

12. Simplified EU Declaration of Conformity

The undersigned NOVAFON – Elektromedizinische Geräte GmbH, declares that the product mentioned in this manual complies with the relevant requirements of the following EU regulations:

- Regulation (EU) 2017/745 on medical devices (MDR)
- Directive (EU) 2015/863 amending the RoHS Directive
- Regulation (EU) 2023/1542 on batteries and waste batteries

The full text of the EU Declaration of Conformity is available at: <https://novafon.com/de/service/downloads#gba>

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- 1 Serritella et al. Local Vibratory Stimulation for Temporo-mandibular Disorder Myofascial Pain Treatment: A Randomised, Double-Blind, Placebo-Controlled Preliminary Study. *Pain Research Management. Pain Res Manag.* 2020 Dec 5;20(2):6705307.
- Staud et al. Attenuation of Experimental Pain by Vibro-Tactile Stimulation in Patients with Chronic Local or Widespread Musculoskeletal Pain.
- Beinert et al. After-effects of neck muscle vibration on sensorimotor function and pain in neck pain patients and healthy controls - a case-control study. *Disabil Rehabil.* 2019 Aug;41(16):1906-1913.
- Guieu et al. Substance P-like immunoreactivity and analgesic effects of vibratory stimulation on patients suffering from chronic pain. *Can J Neurol Sci.* 1993 May;20(2):138-41.
- Lundeberg T. Vibratory Stimulation for the Alleviation of Pain. *The American Journal of Chinese Medicine*, 1984, Summer, 23 (1-4): 60-70.
- Lundeberg T. Long-term results of vibratory stimulation as a pain relieving measure for chronic pain. *Pain.* 1984 Sep;20(1):13-23. doi: 10.1016/0304-3959(84)90807-8
- Lundeberg T. The pain suppressive effect of vibratory stimulation and transcutaneous electrical nerve stimulation (TENS) as compared to aspirin. *Brain Res.* 1984 Mar 5;294(2):201-9.
- Lundeberg T. Relief of pain from a phantom limb by peripheral stimulation. 1985, *Journal of Neurology*; 232(2):79-82.
- 2 Wanderley et al. Rebelatto JR. Effect of plantar vibration stimuli on the balance of older women: a randomized controlled trial. *Arch Phys Med Rehabil.* 2011 Feb;92(2):199-206. doi: 10.1016/j.apmr.2010.10.014. PMID: 21272715.
- NOVAFON GmbH. Results use observation occupational therapy stroke 2020 (unpublished).
- Alashram et. al. Effectiveness of focal muscle vibration on hemiplegic upper extremity spasticity in individuals with stroke: A systematic review. *NeuroRehabilitation.* 2019 Dec 18;45(4):471-481.
- Celletti C, Camerota F. Preliminary evidence of focal muscle vibration effects on spasticity due to cerebral palsy in a small sample of Italian children. *Clin Ter.* 2011;162(5):e125-8.
- Mandic et al. Kinematic analysis of upper-extremity movements after segmental muscle vibration therapy. *Gait & Posture.* 35: S21 - S22. *Gait & Posture*, 2012; 35: 21-22
- Mortaza et al. Upper limb tendon / muscle vibration in persons with subacute and chronic stroke: a systematic review and meta-analysis. *Eur J Phys Rehabil Med.* 2019 Oct;55(5):558-569.
- 3 Johannsen et al. Lasting amelioration of spatial neglect by treatment with neck muscle vibration even without concurrent training. *J Rehabil Med.* 2003 Nov;35(6):249-53.
- Kamada et al. Effects of 5 minutes of neck-muscle vibration immediately before occupational therapy on unilateral spatial neglect. *Disabil Rehabil.* 2011;33(23-24):2322-8.
- Karnath et al. The perception of body orientation after neck-proprioceptive stimulation. Effects of time and of visual cueing. *Exp Brain Res.* 2002 Apr;143(3):350-8
- 4 Barsties V Latoszek B. Preliminary study of Novafon local vibration voice therapy for dysphonia treatment. *Logoped Phoniatr Vocol.* 2020 Apr;45(1):1-9.
- Barsties v. Latoszek. Treatment Effectiveness of Novafon Local Vibration Voice Therapy for Dysphonia Treatment. *Journal of Voice*, Volume 34, Issue 1, 2020, Pages 160.e7-160.e14
- Barsties v. Latoszek B, Watts CR. A Case of Nervus Laryngeus Superior Paresis Treated With Novafon Local Vibration Voice Therapy. *Journal of Voice*, Volume 35, Issue 3, 2021, Pages 406-410.
- Hricová M, Hammer S, Lokale Vibrationstherapie in der Neurorehabilitation - Eignet sich der intraorale Vibrationsspatel zur Verbesserung der Zungenkraft bei Schlaganfallpatienten? - Eine Pilotstudie, *Sprache · Stimme · Gehör* 2021; 45(01): e1-e6
- Yiu et al. Vibrational Therapies for Vocal Fatigue. *J Voice.* 2021 Jan;35(1):29-39.
- Grant L. The use of a manual vibrator in the speech therapy program of four school-age mentally retarded children. *J Commun Disord.* 1982 Sep;15(5):375-83.
- 5 Eklund G, Steen M. Muscle vibration therapy in children with cerebral palsy. *Scand J Rehabil Med.* 1969;1(1):35-7.
- Celletti C, Camerota F. Preliminary evidence of focal muscle vibration effects on spasticity due to cerebral palsy in a small sample of Italian children. *Clin Ter.* 2011;162(5):e125-8
- Grant L. The use of a manual vibrator in the speech therapy program of four school-age mentally retarded children. *J Commun Disord.* 1982 Sep;15(5):375-83.

More Literature can be found on our Website.
Other languages of this manual are available online.